

How do you care for an x-ray apron?

Frequently Asked Questions

- Do not wad, crease, drape or fold your apron or subject it to sharp edges or objects.
- When not in use, store your apron properly on a rack or apron hanger that is made specifically for radiation protective apparel storage.
- See Palmero's assortment of x-ray apron hangers to protect your investment safely.

Hold-It™ X-Ray Hangers

Standard Ref# 28	Economy Ref# 31	CC Style Ref#32	Economy X-Ray Ref# 33	Deluxe Coat Pano Ref# 30
				

- To remove debris from the surface of the **apron**, we suggest using Palmero Healthcare's Protectall™ Wipes which is a convenient cleaner protectant.
- If disinfecting is required we recommend using DisCide® ULTRA surface disinfectant products as recommended prior to Protectall Wipes. Sufficiently wet the surface and allow a drying time of 1 minute to effectively disinfect the surface. Then evenly wipe down the surfaces with Protectall wipes and allow to dry to touch.
- Proper maintenance includes inspecting garments for defects, tears, perforations, or creases.
- It is recommended that x-ray aprons should be tested or inspected annually.



DisCide® ULTRA
Ref# 10DIS/60DIS, 60/160 Towelettes
Ref# 3565Q Quart Spray



Protectall™ Upholstery Cleaner Wipes
Ref# 3800, 50 towelettes

For more information, visit palmerohealth.com,
call 800.344.6424 or email customerservice@palmerohealth.com.



Protecting Clinicians, Patients, and the Practice